Vorderseite

Rosidal® K

100 % CO

Elko® -Rosidal® K

2 Rosidal® K, 10 cm

US: hot

Durelast®

66 % CO, 34 % PA

Durelast® Combi

2 Durelast®, 8 cm und 10 cm

Rosidal® haft

100 % CO

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Short stretch bandages for compression, support and relief bandaging Instructions for Use

Indications:

- Compression bandaging**, for phlebological and lymphological indica- tions, e.g. varices, insufficiency of the perforator veins, thrombophlebitis, phlebotrombosis, post-thrombotic syndrome, chronic venous insufficiency stages I to III, in cases of oedemas (phleboedema, lymphoedema, lipoodema, post-traumatic, postoperative, during pregnancy), after varicose vein procedures.

- Support and relief bandaging** for the prophylaxis and treatment of contusions, distortions, tendon injuries and luxations.

Contraindications:
Advanced peripheral occlusive arterial disease, decompensated heart insuf- ficiency, septic phlebitis, phlegmasia coerulea dolens.

Important notes:

- Rosidal® haft** must be unrolled about 20 cm prior to application.
- Elko®-Rosidal® K** - For compression bandaging in double bandage tech- nique (fig. A), the second **Rosidal® bandage** must be applied over the first one in the opposite direction using the same technique. Start either at the ankle joint or at the base of the toes (see fig. A10 and A11).
- Durelast® Combi** may be applied according to Sigg's compression bandag- ing technique (fig. B). To start bandaging, use the 8 cm wide **Durelast®**. From the calf onwards, the 10 cm bandage is applied.

General notes:

- Ideally apply the bandage when the swelling is down, preferably after the patient has just got up in the morning or after having rested the leg in a raised position for some time.
- Select a bandage which is equal in width to the diameter of the limb to be treated. Bandages, 8, 10 or 12 cm wide, are most commonly used.
- Unless otherwise indicated start bandaging on the foot which, should form an angle of 90° with the leg.
- Apply the bandage at consistent pressure to ensure graduated compres-

- sion from the foot to the knee.
- Apply the bandage holding the roll close to the limb and pointing outward.
- Ask the clinician to indicate the particular bandaging technique he or she wishes to be applied.
- Inexpert bandaging may lead to cutaneous gangrene and nervous pressure injuries.

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